

High marks for performance, no marks on the bridge of the nose

Innovative under-the-nose design

prevents red marks on the bridge of the nose and gives your patient the widest field of vision of any of the leading full-face masks*

Easy to use

with one modular frame for all cushion sizes

Small cushion	1090692
Medium cushion	1090693
Large cushion	1090694

Quick release clips

for easy on and off

Soft, comfortable fabric straps

Large headgear	1090696
Standard headgear	1090697

Quick disconnect

Quick release tube	1090689
--------------------	---------

Headgear clips x 2	1090698
--------------------	---------

Mask with headgear	International ⁽²⁾	Global ⁽¹⁾
Small	1090662	1090602
Medium	1090663	1090603
Large	1090664	1090604

Sizing gauge

1090699

*internal data on file..

Helpful tips

Here are some tips to keep in mind when fitting and using the Amara View sleep therapy mask.



Unlike most full face masks, the cushion size for Amara View is determined by measuring the depth of the nose using the simple sizing gauge.

Just place the sizing gauge under your nose **without pushing against the cheeks** and the tip of the nose will indicate whether they need a small, medium or large cushion size. **If you measure close to the threshold always choose the smaller size.**

Fitting instructions



Loosen the headgear. Twist the headgear clips to disconnect.



Press the headgear clips into place to connect.



Choose the cushion size that covers the mouth and sits at the nose base.



Pull back the headgear tabs to evenly adjust the top and bottom straps.



Gently holding the mask against the face, pull the headgear over the head.



Adjust the crown strap so the headgear sits loosely and comfortably.



Ensure the nose is over **(not inside)** the cushion nostril opening.



Connect to a working CPAP machine to make final adjustments.

For more fitting tips, find out how to adjust for leaks or cleaning instructions, please take a look at our video guides, visit: www.sleepapnea.com/products/masks/

(1) Global: Brazilian Portuguese, Dutch, English, French, German, Italian, Spanish.

(2) International: Arabic, Bulgarian, Croatian, Czech, Danish, Dutch, English, Estonian, Finnish, French, German, Greek, Hebrew, Hungarian, Italian, Kazakh, Latvian, Lithuanian, Macedonian, Norwegian, Polish, Portuguese, Romanian, Russian, Serbian, Slovak, Slovenian, Spanish, Swedish, Turkish, Ukrainian.

© 2015 Koninklijke Philips N.V. All rights reserved.
Specifications are subject to change without notice.

www.philips.com/respironics
Customer Service +1 724 387 4000
PN 1126893 MCI 4107168

