

Portable Unit Filling Procedures

1. Clean the reservoir AND portable unit fill connectors to remove moisture and other contaminants with a clean dry towel.
2. Position the portable above the quick disconnect valve and align it with the indentation at the top. Press down on top of portable to engage the unit.
3. Once the unit is engaged, filling can begin by pulling down the vent valve into its open position while applying pressure to top of unit to keep it in place.
4. After 20-30 seconds of filling, open and close the valve a few times...this will help keep the valve from sticking.
5. Unit is full when hissing sound changes in tone and a dense white vapor is observed. Close the fill lever by lifting it up.
6. Disengage the unit by hitting the disconnect plunger on the reservoir.
7. If the units freeze together, close the vent valve and wait till the connectors have thawed. They will then be easy to separate.